

## Catalogue Number

### Reusable range

KM218800	Hospital NV Full Face Reusable mask - Size S
KM218900	Hospital NV Full Face Reusable mask - Size M
KM219000	Hospital NV Full Face Reusable mask - Size L
KM222400	Disposable headgear X10

## Compatibility with ventilators

	Monnal T50	Monnal T60	Monnal T75	Extend	Other valve ventilators
Compatibility	✓	✓	✓	✓	✓



## Contact

Air Liquide Medical Systems - Parc de Haute Technologie  
 6, rue Georges Besse - 92182 Antony cedex  
 Tel.: +33 (0)1 40 96 66 00 - Fax: +33 (0)1 40 96 67 00



## Hospital NV Full Face Reusable mask

The solution to benefit from our comprehensive expertise in non-invasive ventilation




Safety

Efficiency

Environment-friendly


- Provides non-invasive, comfortable ventilation
- Guarantees complete safety for the user
- Protects the environment

# Hospital NV Full Face Reusable mask

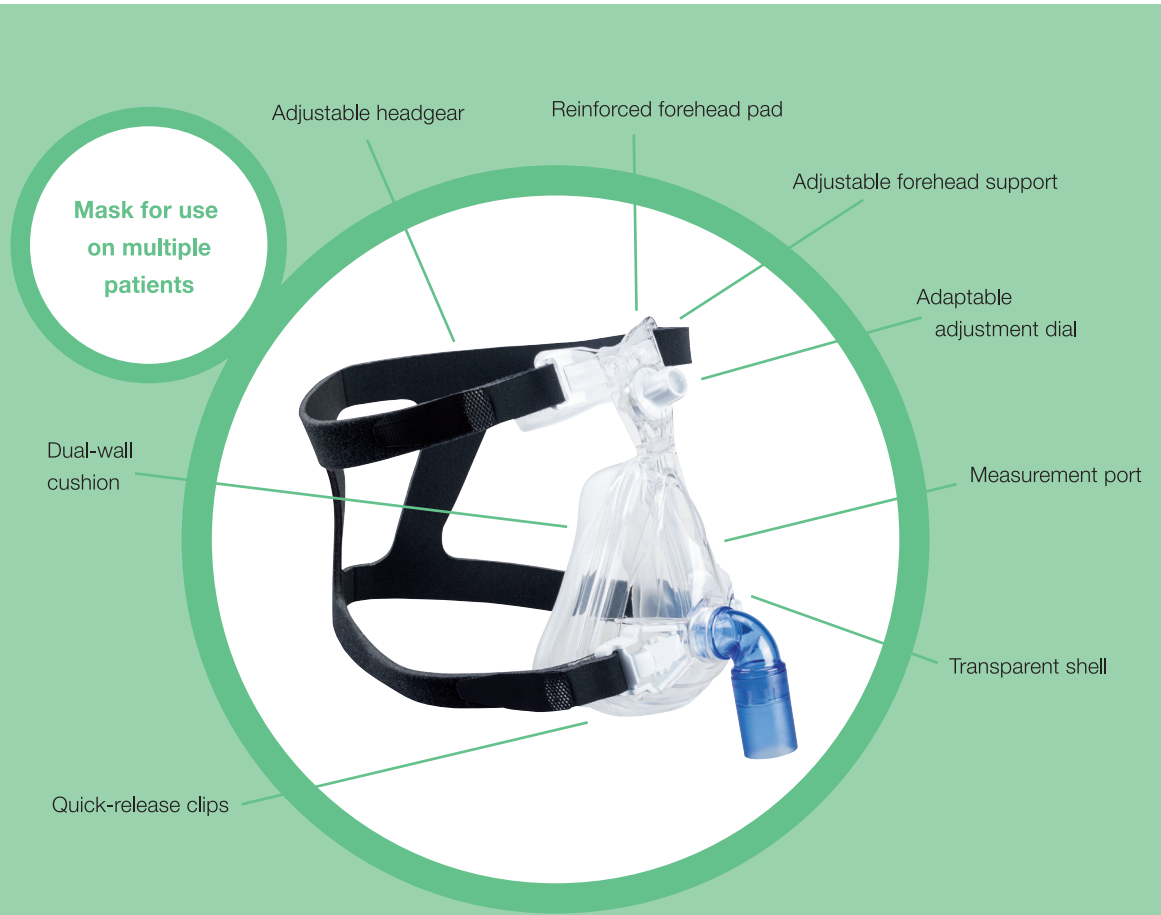

- Mask available in 3 sizes
- Sizing-gauge attached on the packaging
- Package includes mask, cushion, and headgear assemblies

## Fitting recommendations

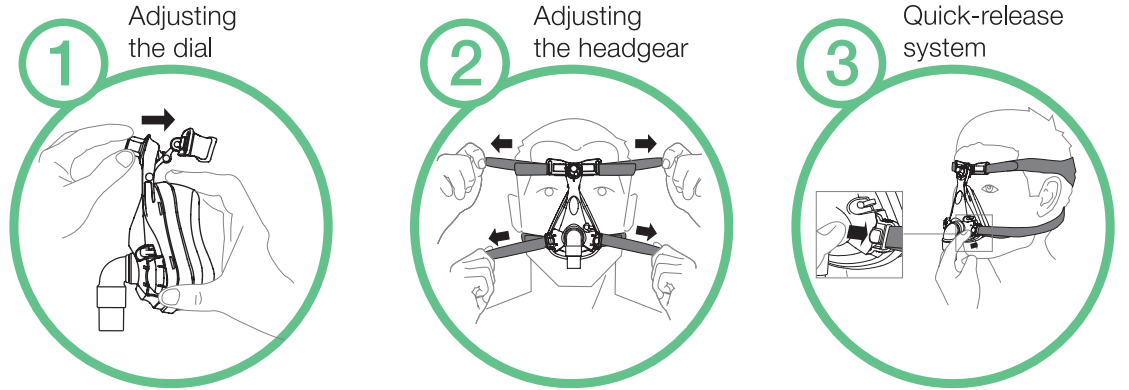

### Decontamination

- Aniosyme DD1, detergent with neutral pH


### Disinfection

- High-level thermal
  - Aniosyme DLM Maxi, detergent with neutral pH
  - 93°C (200° F) for 10 minutes
- High-level chemical
  - Aniosept Activ, Anioxyde 1000, Cidex Opa, Opaster Plus

### Sterilization

- Chemical
  - Sterrad 100S/NX/100NX, Autoclave
- Autoclave
  - 121°C (250° F) for 15 minutes

Number of cycles authorised: 20

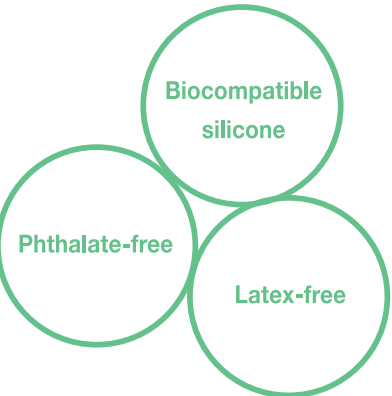

For the complete disinfection and sterilization protocols, please refer to the document available on our website [www.airliquidmedicalsystems.com](http://www.airliquidmedicalsystems.com)